



Soredex Optime TWAIN Installation Instructions Version: 11.10

Date Created: 1/3/05 Last Modified: 2/11/05 Filename: Optime TWAIN Installation.doc

Re	equired materials:	Re	quired EagleSoft modules:	Ins	stall on the following computers:
•	Digora for Windows	•	Integrated Digital X-Ray,	•	Any computer that will access
	preinstalled		Video Capture, or Both –		the Optime
٠	Optime TWAIN drivers CD		Access to Advanced		
•	Optime(s)		Imaging		
•	Power cable				
•	Network cable				

*The Soredex Optime is a network configurable device. It must have it's own static ip address on the client's network

*Digora for Windows must be preinstalled for Eaglesoft v11.10 in order to configure the ip address for the Optime.

INSTALLING DIGORA FOR WINDOWS:

1. Install the Digora for Windows CD



2. Click Next

cense Agreement			ALC: NOT
Please read the following license agreeme	nt carefully.		25
Press the PAGE DOWN key to see the res	st of the agreement.		
SOFTWARE LICENSE			<u>_</u>
THIS DOCUMENT IS A LEGAL AGREEM AND SOREDEX (THE <licensor>) GO DIGORA FOR WINDOWS SOFTWARE (</licensor>	IENT BETWEEN YOU VERNING YOUR RIG «Software»).	J, AS THE «LIC HT TO USE TH	ensee>, He
IF YOU AGREE TO BE BOUND BY THE SOFTWARE LICENSE AND THIS PARAI PROMPT BELOW IN WHICH CASE YOU ENFORCEABLE LIKE ANY WRITTEN NE	TERMS AND CONDI GRAPH, PLEASE PRI J AGREE THAT THIS	TIONS OF THE ESS <i agree=""> / AGREEMENT</i>	AT THE IS
Do you accept all the terms of the precedir will close. To install DfW2.5B 67 Single, yo	ng License Agreement ou must accept this ag	? If you select preement.	No, the setur
lahreid:	-		

3. Answer Yes.



4. Click Next



5. Leave all boxes checked by default. Click Next.

DfW2.5B 67 Single - InstallShield Wizard			×
Select Options Select the options you want to install.			X
To install a feature, click the check box next t be installed.	to it. If the check	box is clear, that	feature will not
DIGORA PCT			
C SCANORA D			
InstallShield			
	< Back	Next >	Cancel

6. Leave the option default to DIGORA PCT even though that is not the device to be installed. Click **Next**.



7. Click Next

DfW2.5B 67 Single - InstallS	hield Wizard
	InstallShield Wizard Complete Setup has finished installing DfW/2.5B 67 Single on your computer.
	Kack Finish Cancel

Page 3 of 6

INSTALLING THE TWAIN DRIVER FOR THE SOREDEX OPTIME:

- 9. Install the Soredex Optime TWAIN Drivers CD
- 10. The **Digora Scanner Driver** will have to be installed before loading the TWAIN device for the Optime.

	Welcome to the Digora Scanner Driver Setup program. This program will install Digora Scanner Driver on your computer.
	It is strongly recommended that you exit all Windows programs before running this Setup program.
	Click Cancel to quit Setup and then close any programs you have running. Click Next to continue with the Setup program.
	WARNING: This program is protected by copyright law and international treaties.
-	Unauthorized reproduction or distribution of this program, or any portion of it, may result in severe civil and criminal penalties, and will be prosecuted to the maximum extent possible under law.

11. Click Next.



12. Click Next



- 13. Leave the option at **Yes** to reboot the computer and click **Finish.**
- 14. After the reboot, access the CD again. This time, load the **TWAIN device** for the OPTIME.



15. Click Next

Ustomer Inf Please enter	ormation your information.			
User Name:				
PTC				
Company Na	me:			
PTC				
Install this ap	plication for:	is computer fall us	ers)	
	C Only for me (USER)			
anchaster -				
anonancare -				

16. Enter the Client Information and click Next.



17. Click Finish.